


# Quick Guide to the CR110 Remote Assistant

## Performing a Daily Check




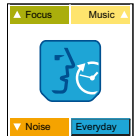
### Switch processor on

Hold down the **Lower Button** for two seconds (LONG hold).

- ▶  LED on processor flashes and the number of green flashes indicates current programme


### Switch remote assistant on

Hold down the **Cochlear Home Key** for three seconds (LONG hold). Note: make sure slider on the right hand side is in Up position (unlocked).

- ▶  →  Cochlear symbol appears in LCD screen and then changes to the Home screen

### Go to Status screen


Quick press on the **Cochlear Home Key**.

- ▶  Icons appear to show if any parts are faulty or if the coil is off the implant (press Cochlear Home Key to clear).



Green Processor icon will appear to show all is working well.

### Talk into the sound processor microphone

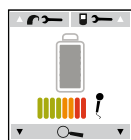

Screen will automatically turn to Status screen. Talk into the sound processor microphone and watch input change on the bars.

### Check battery status of sound processor



Check screen to see if Battery icon is full (grey) or empty (white).

### Put coil on back of remote assistant



Line up the coil with guides on back of the remote assistant. Watch for transmission signal on LCD.

### Plug in Freedom™ listening earphones

Using the Freedom accessory adaptor.

- ▶ Listen to the sound quality of the microphone.

Should any problems persist please refer to the User Guide or contact your CI centre

Phone \_\_\_\_\_