





Using an accessory and/or the monitor earphones

Accessories:

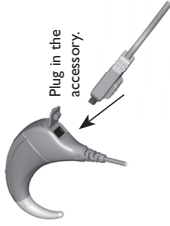

Accessories and earphones:

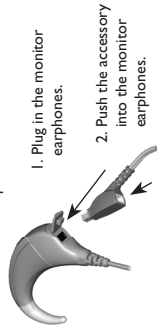

1. Plug in the monitor earphones.
2. Push the accessory into the monitor earphones.

Lock and unlock

Press the Select button and the Decrease button to lock or unlock the processor.



Changing volume

1. Briefly push either the Increase or Decrease buttons.
2. Push again, for a slightly longer time, on either the Increase or Decrease buttons until you hear a beep and/or see a flashing 'V' on the display.
3. Continue briefly pushing either the Increase or Decrease buttons until you reach the volume level you want.

Using the in-built telecoil

Push both the Increase and Decrease buttons until 'T' shows on the display.



Resetting the speech processor

Press all three buttons (Select, Increase and Decrease) together (for at least one second) to return to the default settings.



Mic lock

Attach the microphone lock as shown.



Help message	Meaning	Action
H1	Flat battery	Change all batteries. Note: you will stop hearing sound and only be able to turn off the speech processor.
H2	Low battery	Change all batteries.
H3	Coil error	Check: <ul style="list-style-type: none"> • coil position • cable connections • for cable damage.
H4	Audio error	Follow the steps in the Troubleshooting section of your User Manual to try to resolve the problem.
H5	Program corrupted	Use an alternate program where it is available until you are able to return the speech processor to your clinician. They can install a new program for you.

